

The Board of Pharmacy has received notice of the following product recall. The Board strongly encourages pharmacies to immediately review their quality assurance and recall policies and procedures to determine if any corrective action is required.

Roche is recalling Accu-Chek® Inform II Controls Lot Number 30100519 — Level 1 due to the control barcode being unable to scan due to the barcode being cut off. Customers are unable to scan the Accu-Chek Inform II level 1 control barcode due to the barcode being cut off. The barcode contains the encrypted lot number that is created using the human readable lot number printed on the control bottle and the control level. Accu-Chek Inform II customers can scan the barcode for the control lot number and level, or they can manually select the control level and enter the lot number on the meter (if configured appropriately). Human readable information is printed correctly on the control bottle and the correct level is contained in each bottle. The issue does not affect the entire control lot. If another box of controls with the same lot number is available, the customer can attempt scanning a level 1 bottle from that box.

PRODUCT: Accu-Chek Inform II Controls

INSTRUMENT: Accu-Chek Inform II Meter

LOT NUMBER: 30100519

CATALOG NUMBER: 05213509001